

HF-B51

DIGITAL BLOOD PRESSURE MONITOR

Take care of your health



Eight characteristics Concern about your need



Display blood pressure



One click measurement



HD screen with backlight



2*99 groups memory



Self-measure with cuff



Voice broadcast



Remind arrhythmia



Divide into A/B groups

Product Details

Model	HF-B51	
Operation	Temperature:5~40°C/41-104°F	
	Humidity:15%~85%RH	
Storage	Temperature:-20°C~55°C / -4 ~ 131°F	
	Humidity:10%-95%RH	
N.W	About 650 g	
Product	138(L)×105(W)×74(H)mm	
dimensions		
Measuring Range	Blood Pressure:0 to 280 mmHg(0 to 37.3 kPa)	
	Pulse Rate:40 to 195 times/minute	
Service life	5 Years (Without Battery)	
Display Mode	Digital liquid crystal display	
Accuracy	Blood Pressure: ± 3mmHg /±0.4kpa	
	Pulse: ±5%	
Power Supply	4*AA Alkaline battery/DC adapter	
Enclosure	Monitor*1,certificate*1,Battery*4,Cuff*1,user manual*1	



One Key Operation

One Button measurement of blood pressure, automatic completion of the whole process of measurement





2*99 groups memory

 $2\,{}^*\,99$ groups , automatic storage of measurement resulets , more convenience than manual storage







IHB HEART RATE TEST


In the measurement of blood pressure at the same time one the heart rate is also tested, a machine is with multi-function





SMART VOICE BROADCAST (OPTION)-SUPPORT YOUR LOCAL LANGUAGE

Support your local Language, for the Elderly with known date difficulties. voice broadcast can convey blood pressure information more conveniently

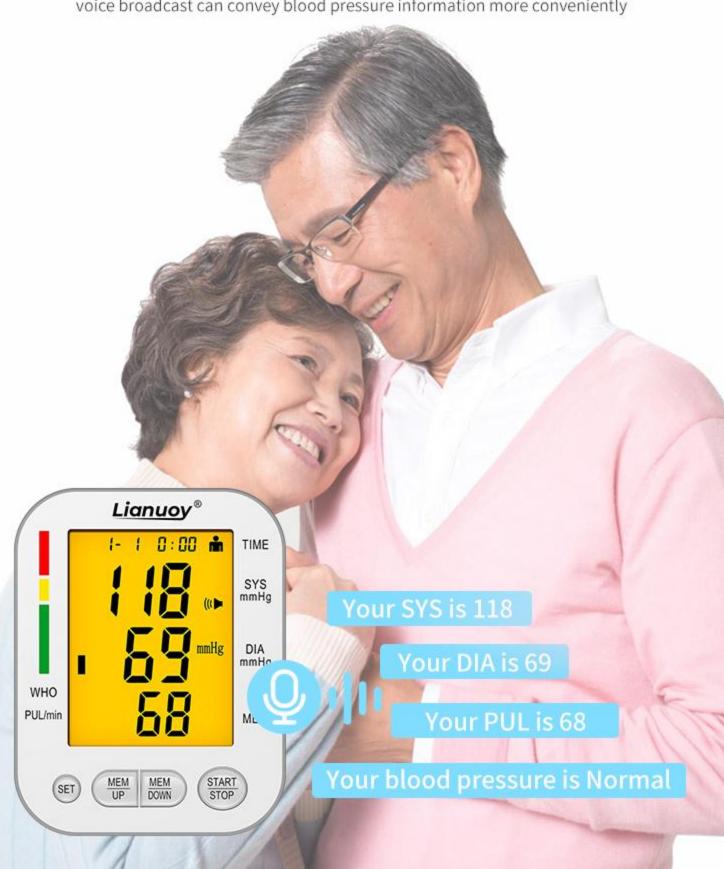



HD screen With backlight


EASY TO READ EVEN IN THE NIGHT





BLOOD PRESSURE CLASSIFICATION (WHO) INDICATOR

Blood pressure classification (WHO) indicator

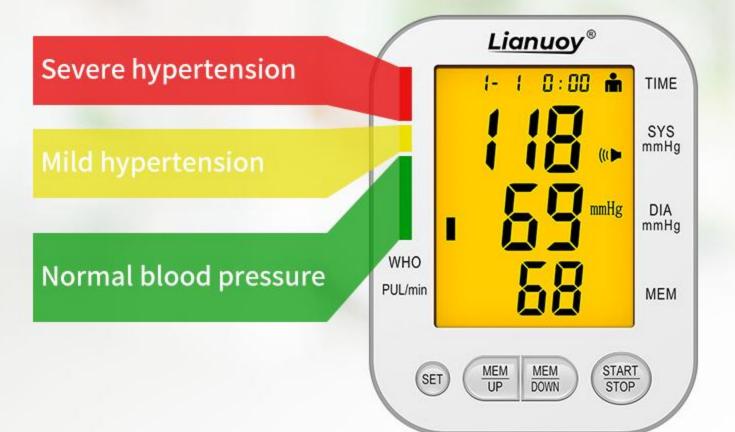



ANTIBACTERIAL ARM BELT

Measuring safety and health





DUAL POWER MODE

Using Battery or Plugging in






Battery Type-C






YOU CAN DO IT AS NEEDED TURN SOUND ON OR OFF

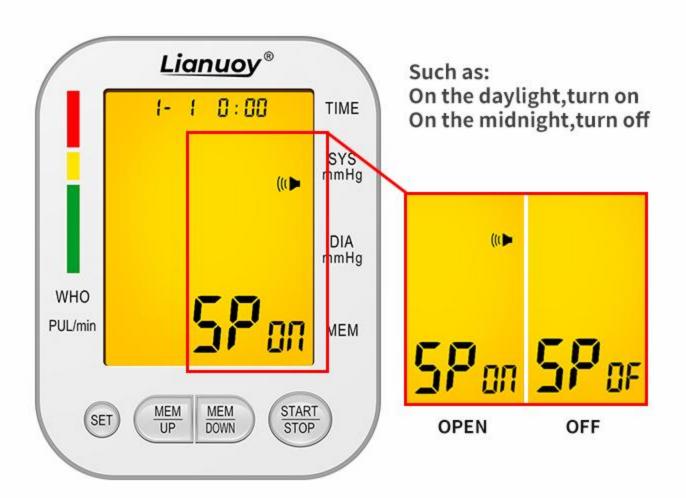



AUTOMATIC POWER OFF AND LOW BATTERY REMINDE

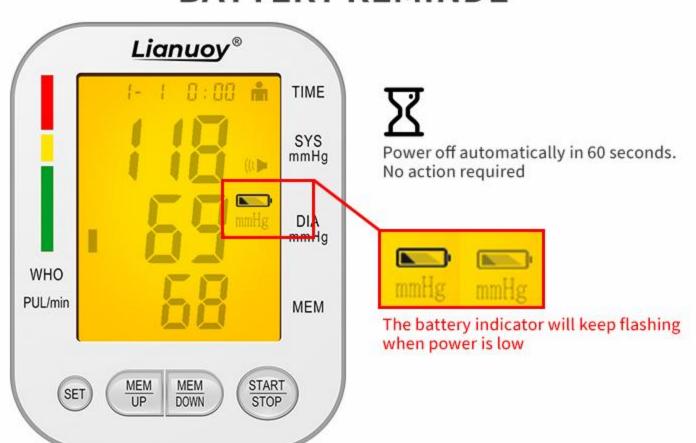

OPERATING INSTRUCTIONS


Correct posture and precautions (before the measurement, please rest for at laest 15 minutes and maintain a relaxed and)

Corrrvt posture

Measurement of blood pressure Both the left arm and the right arm can be measured.

Go to the toilet in advance.

Please perform the measurement in a room with a suitable temperature.

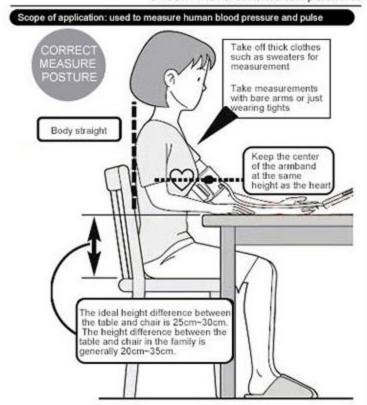

Step 1:

Sit and relax

Step 2:

Place the armband on the left arm (2-3cm above the elbow joint), close to the skin and wrap the air tube in the middle (middle finger extension line)

Step 3:

After the cuff is rolled, a finger should be freely inserted between the cuff and the arm

Step 4:

The middle of the armband must be at the same level as the heart

Step 5:

You can put your hands on your thighs and relax your palms upwards naturally





- Sit cross-legged when measuring.
- Bend down (leaning forward)
- ➤ Sitting on a sofa or a low table causes the body to bend (because the negative pressure is generated or the position of the arm is lower than the heart and the blood pressure rises), the ideal height difference between the table and the chair is 25~30CM. If necessary, put a soft cushion under the arm.

CERTIFICATE



Debe the line

量(任豆





-

- MI F

COMPANY PROFILE

Welcome to our company





Shenzhen Hopfat Medical Instrument Co. Ltd was established in 2010, located in Baoan District , Shenzhen, China. Hopfat is a professional global medical device OEM/ODM manufacturing supplier in researching, manufacturing and exporting which mainly engaged in digital electronic Blood pressure monitor, Digital electronic thermometers, Non-contact forehead thermometer and other medical devices. The company has passed CE, RHOS, FCC and other certifications. At the same time; it has more than 10 patents and a number of software copyrights; the factory strictly in accordance with ISO13485 quality management system for production management, the current products have been exported to more than 50 countries and region. Such as United States, Netherlands, Kenya, Vietnam.

Hopfat has a professional design team who make design as customer's requirement, core software algorithms and clinical improvement capabilities; it also has completely automatic equipment and multiple assembly lines, the annual maximum production capacity of electronic blood pressure monitor is 2.4 million units, the infrared

thermometers is 72 thousand units, and the Non-contact forehead thermometer is 1.2 million units; Fast shipment. Will ship the goods in 3-5 days under 500 pcs. Will ship the goods in about 2 weeks

About After-service, Support 1-2 years quality guarantee in difference goods. We will be patient to answer or solve the questions


Trust and Choose us! Take care your health and contact us!

OUR SERVICES



OEM/ODM

- 1 OEM your native language. Such as Japanese, Korean, German, Spanish, Russian and so on.
- 2 Embossing your logo. Send your logo file to us, then we will make desing for your reference and then start to producdtion after confirm.

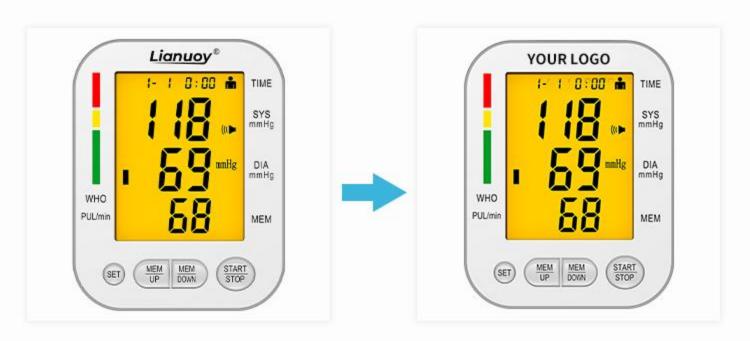

Order goods with our brand

Support to send some promotion or attractive pictures to you for publicity

OUR TEAM

Product R & D department









OUR PRODUCTION LINE

Product quality inspection and assembly department







QUALITY CHECK SCENARIO









PACKAGE DETAILS







Package size:410*400*360mm Product Model:HF-B51

Quantity:24pcs/carton

GW:16KG




TRANSPORTATION DETAILS









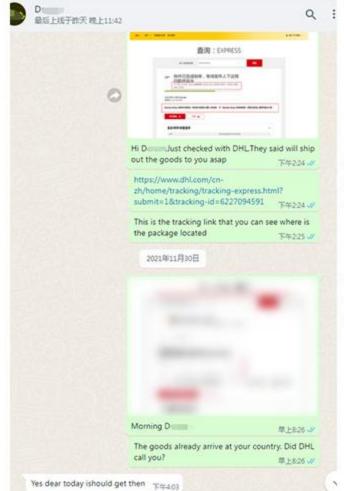

Ship by our agent (DAP/CIF term):

- 1. Ship out the goods,
- 2.Send the Tracking number and Tracking link to you,
- 3. Remind you when you are about to receive the goods.

Ship by your agent in China (FCA term):

- 1.Ship out the goods,
- Send the Tracking number and Tracking link to you
- 3.Remind you that your Chinese forwarder have already received the goods.

FAQ

- 1. Are you Manufacturer or Trade company?
- ---We are a Manufacturer. You can check our factory video on Alibaba or you can video call with us to visit our factory.

2.Why choose us?

We are factory, can provide reasonable price and good after-service. Provide OEM&ODM service, Such as Customized your native language, Embossing your logo on items or change the outlook as your requirement.

- 2.What's certificate you have?
- --- CE, FCC, RHOS and ISO13485.
- 3.What's your MOQ? Could I order some samples at first time?
- ---The MOQ is 24pcs for Blood pressure monitor. The MOQ is 240pcs for digital thermometer.

As our support, you can order some samples first to check the quality and market if you want.

- 4.What's your payment terms?
- ---Accept TT(Bank transfer), Trade Assurance or others.
- 5.What's your quality guarantee period?
- --- 1 year from shipment.
- 6.What's the sample lead time&production lead time?
- ---Will ship out in 2 days for samples. The production time is about 5-15days for bulk order.
- 7.Do you support OEM/ODM order?
- ---Sure. Just contact or call us. Then we will make the goods as your requirement.

CONTACT US





+86 18124768286



Sales6@hopfat-w.com



www.lianuoyhealth.com



